Welcome to MEO™

EN MEO™ is the world's most innovative anti-pollution mask, bringing together the most advanced materials and the brightest designers – so not only do you breathe cleaner air, you look great too.

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EN

WEARING INSTRUCTIONS

1. Face mask outward. Hold straps.
2. Put on the face mask, your chin should be in the mask and straps should be under the ears.
3. Pull the straps backward and secure by using the clip.
4. Adjust the face mask to make it fit the face.

RECOMMENDED FILTER LIFE

The filter should be replaced every 3 weeks. Where pollution is more extreme, filters may need to be replaced more frequently.

STORAGE

1. **Storage Environment:** ventilation, cool, dark places. Keep away from fire and contaminated areas.
2. **Storage Temperature:** -20°C~40°C less than 80% humidity.

The filter is not washable.

The mask outer facing can be hand washed gently in warm water.

The mask can filter some pollutants, however, it cannot eliminate the risk of illness and infections.

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
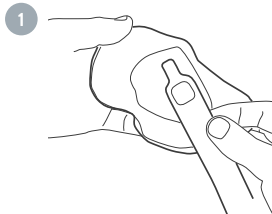
EN Please make sure the pointed part of the mask skin is facing up. This is the part that touches the nose. Have the Inner liner facing you as you prepare to install the left strap.

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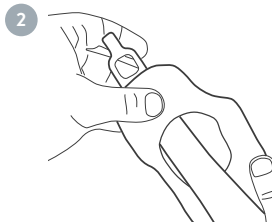
EN Slide the Left strap through the slot in the left side of the mask outer.

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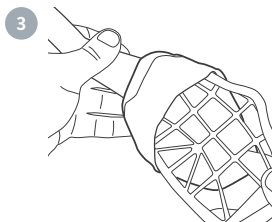
EN Slide the skin down the strap until the slot is positioned over the end of the strap where it attaches to the frame.

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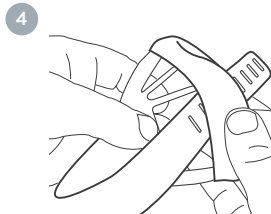EN Take the right strap and feed it through the remaining slot in the mask skin.

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EN Slide the cover down the strap. Pull the liner over the edge of the whole way around the frame. Position the seam of the mask skin at the edge of the mask around the entire edge.

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EN Pull the cover down the strap until the slot is positioned over the end of the strap where it meets the frame. Some adjustment may be necessary to get the outer into position.

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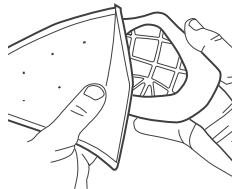
1


EN Push the filter into the mask cavity.

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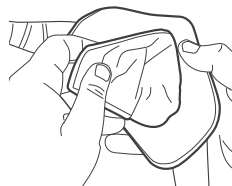2

EN Ensure the seam is at the front of the filter towards the top.

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3

EN Carefully position the edge of the filter into the sealing cavity that surrounds the frame.

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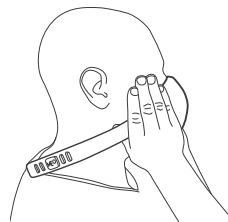
NOTE: The filter should be able to be replaced without pulling the mask skin off, however we recommend that the user washes the mask skin at the time of filter replacement.

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1


EN Place mask against the face firmly using your hands at the base of the straps.

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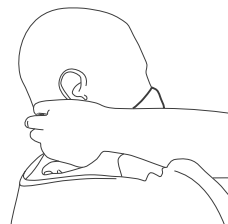2


EN Keep tension of the mask against the face while sliding your fingers towards the back of your neck.

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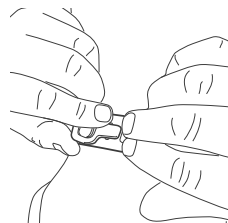3

EN Slide loop over the clip. Adjust the strap so that there is sufficient tension on the straps to keep the mask on your face.

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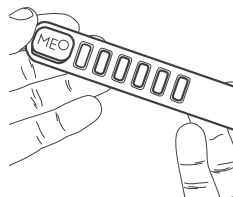
CLIP ADJUSTMENT

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1


EN Take the adjustment strap in your hands with the MEO™ logo facing up.

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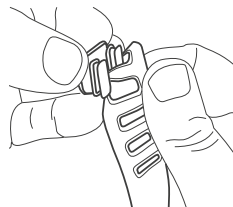2


EN Pull the clip out of the strap. Try mask on without clip and overlay straps to determine where the clip needs to go.

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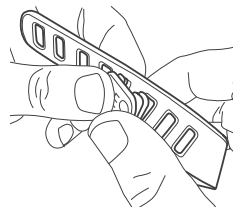3

EN Push the clip into the strap at the desired position.

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Healthy Breath Limited