MEO™

Welcome to MEO™

EN

MEO™ is the world's most innovative anti-pollution mask, bringing together the most advanced materials and the brightest designers – so not only do you breathe cleaner air, you look great too.

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EN

WEARING INSTRUCTIONS

1. Face the mask outward. Hold the straps.
2. Put on the mask. Your chin should be in the mask and straps should be under your ears.
3. Pull the straps backward and secure them by using the clip.
4. Adjust the mask to make it fit your face.

RECOMMENDED FILTER LIFE

The filter should be replaced every three weeks. Where pollution is more extreme, filters may need to be replaced more frequently.

STORAGE

1. **Storage Environment:** store in a well ventilated, cool, dark environment. Keep away from fire and contaminated areas.
2. **Storage Temperature:** $-20^{\circ}\text{C}\sim 40^{\circ}\text{C}$, less than 80% humidity.

The filter is not washable.

The mask facing can be hand washed gently in warm water.

The mask can filter some pollutants, however, it cannot completely eliminate the risk of illness and infections.

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
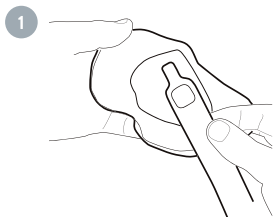
EN Please ensure the pointed part of the mask facing is pointing up. This is the part that touches your nose. Make sure the inside of the mask facing is facing you as you prepare to install the left strap.

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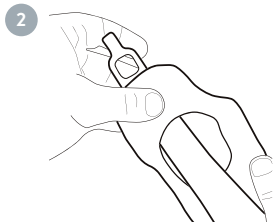
EN Slide the left strap through the slot in the left side of the mask facing.

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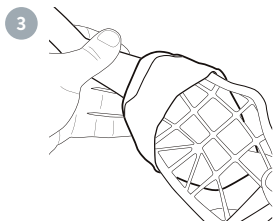
EN Slide the mask facing down the strap until the slot is positioned over the end of the strap where it attaches to the frame.

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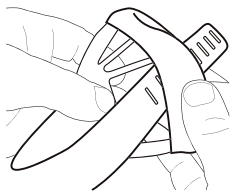4


EN Take the right strap and feed it through the remaining slot in the mask facing.

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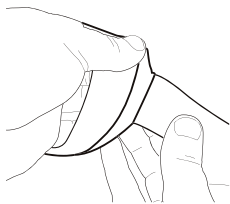5


EN Slide the mask facing down the strap. Pull the mask facing over the frame edge, all the way across the entire frame, so that the seam of the mask facing sits along the edge of the frame.

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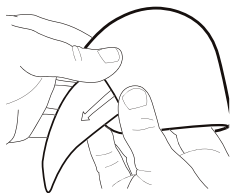6

EN Pull the mask facing along the strap until the slot is positioned over the end of the strap where it meets the frame. Some adjustment and manipulation of the mask facing may be necessary to get it into position correctly.

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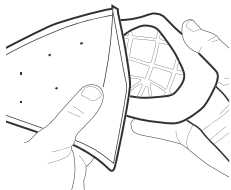
1


EN Push the filter into the mask cavity.

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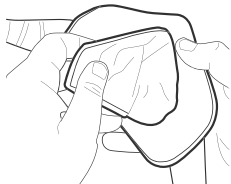2


EN Ensure the seam is at the front of the filter towards the top.

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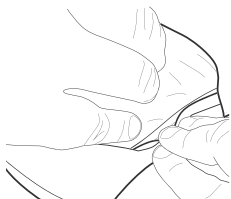3

EN Carefully position the edge of the filter into the sealing cavity that surrounds the frame.

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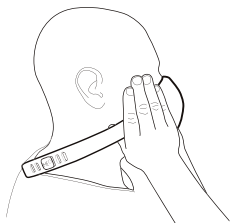
NOTE: The filter should be able to be replaced without pulling the mask facing off, however we recommend washing the mask facing at the time of filter replacement.

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1


EN Firmly place the mask against your face, holding onto either side.

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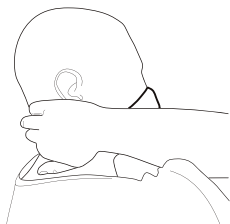2


EN Keep the tension of the mask against your face while sliding your fingers towards the back of your neck.

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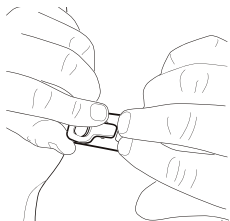3

EN Slide the loop over the clip. Adjust the strap so that there is sufficient tension on the straps to keep the mask on your face.

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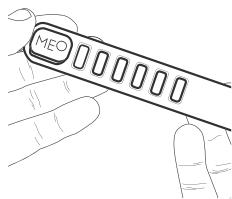
CLIP ADJUSTMENT

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1


EN Take the adjustment strap in your hands with the MEO™ logo facing up.

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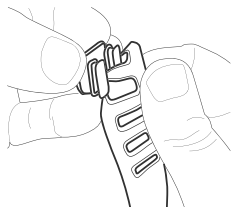2


EN Pull the clip out of the strap. Try the mask on without the clip and overlay the straps to determine where the clip needs to go.

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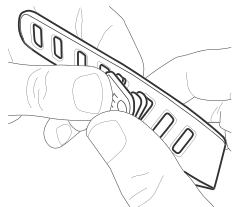3

EN Push the clip into the strap at the desired position.

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Healthy Breath Limited